

Squared Toe Post-Op Shoe

PATIENT APPLICATION



BEFORE USING THIS DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USE/INDICATIONS:

For use following surgery or trauma to the foot.

CONTRAINDICATIONS:

None

WARNINGS AND PRECAUTIONS:

If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

APPLICATION INFORMATION:

1. Unfasten ankle strap and forefoot closure. Slip foot into the shoe.
2. Secure both strap and forefoot closure for a snug but comfortable fit.

CLEANING INSTRUCTIONS:

To clean shoe, wipe with a damp cloth and air dry.

WARRANTY:

DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

FOR SINGLE PATIENT USE

NOT MADE WITH NATURAL RUBBER LATEX

NOTICE: While every effort has been made in state-of-the-art techniques to obtain the maximum compatibility of function, strength, durability and comfort, there is no guarantee that injury will be prevented through the use of this product.

