ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS:

The Procure Surround Ankle w/Gel Brace is designed to support the ankle. It may be suitable for use following light to moderate ankle sprains, and help with Chronic Ankle Instabilities (CAI). Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

CONTRAINDICATIONS: N/A

APPLICATION INFORMATION:

- 1) Open ankle stirrup so gel and closed cell foam are facing upward. (Figure 1)
- 2) Position the heel evenly on the center pad. Wrap lower hook and loop closure strap around ankle for alignment. Center side shells along ankle and leg. Wrap upper strap and adjust if necessary. (Figure 2)
- 3) Apply pressure to the sides of the therapeutic gel bag to allow even flow of pressure within the plastic shell. Tighten lower and upper straps for a secure fit and patient comfort. (Figure 3) Straps may be trimmed if necessary.
- 4) Adjust vertical side straps, placing both hands on straps, pulling upward until desired fit. (Figure 4) Secure vertical side straps with hook and loop closure.

COLD THERAPY

- Remove therapeutic gel bag from ankle support system and place in freezer for 2 hours
- Place gel bag in ankle support and follow fitting instructions.

NOTE: absorbent sock should be worn before ankle support is fitted

CLEANING INSTRUCTIONS:

- Outer shell and gel bag may be hand washed with warm water and mild detergent. AIR DRY.
- Freeze only therapeutic bag. DO NOT place entire system in freezer.

WARNINGS AND PRECAUTIONS:

- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- Do not use over open wounds.
- Do not use this device if it was damaged and/or package has been opened.
- If you develop an allergic reaction and/or experience itchy, red skin after coming into contact with any part of this device, please stop using it and contact your healthcare professional immediately.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

FOR SINGLE PATIENT USE ONLY.

 NOT MADE WITH NATURAL RUBBER LATEX.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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