


INSTRUCTIONS FOR USE STERISHIELD II™ DUAL LAYER PROTECTIVE AMNION PATCH




Stores at room temperature. No refrigeration necessary.



Wait to open inner package until ready to place graft.

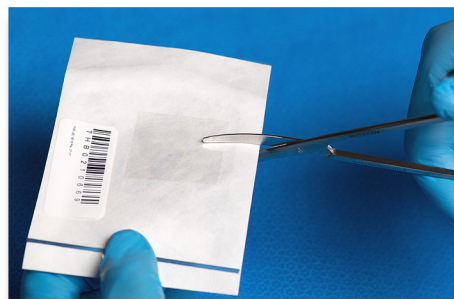


Trim through the sterile inner package if needed.

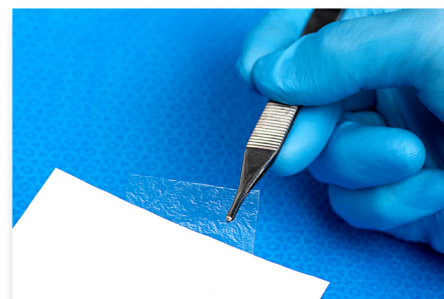

Take graft directly from package to surgical site.



Apply dry, ideally with forceps.

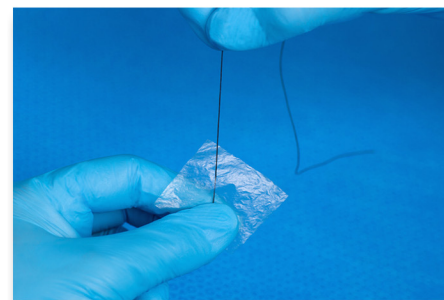

Sutures or adhesives may be used if indicated.

Other Comments:

Apply in wound environment with either side facing the wound; dual layer version eliminates side specific placement issues of other amnion products. Can be manipulated when in the wound site; if graft becomes folded or difficult to handle, remove and place in saline (or any other sterile solution), as it will flatten out to original state.