



Ride the wave of Re-form

REFORMABLE, ADJUSTABLE BRACING SYSTEM



MOTION⁺ IS MEDICINE

Open it, Close it

Exos is the **leading** bracing system offering you and your patients a removable, adjustable, reformable and waterproof solution for the treatment of fractures and other injuries requiring stabilization.

REMOVABLE

Although the Exos brace is custom fit to your patient, it can be removed, allowing:

- Examination of the fracture site
- Examination of a wound or an incision
- Assessment of muscle atrophy
- The ability to initiate rehabilitation therapy earlier in the healing process
- Skin and brace cleaning
- Sawless removal

Comfortable, lightweight material made from



Exclusive Boa® dynamic closure system

Antimicrobial treated

Washable and waterproof



To ensure compliance, the Boa Locking Ring prevents unprescribed loosening or removal.



1. Simply heat brace in our oven for 3-5 minutes.



2. Open and slide onto patient's limb.



3. Gently mold and adjust for a comfortable fit.



4. Use the reel and lace to tighten as needed.

REFORMABLE

The Exos brace is completely reformable, allowing:

- Modification to accommodate anatomical changes as healing progresses
- Serial changes of position
- One brace throughout duration of treatment
- Less material waste

Warm it, Fit it

ADJUSTABLE

Exos uses the exclusive Boa reel and lace dynamic closure system, allowing:

- Expansion to allow for initial swelling and to reduce the risk of Compartment Syndrome
- Tension modification for custom fit despite atrophy or changes in soft tissue
- Improved patient comfort and fit



It Takes to Water

The Exos brace is waterproof,* allowing:

- Patients to continue their regular daily activities such as showering
- Children to remain active
- Athletes to continue water activities
- Use of blow dryer to quickly dry brace and skin

It's Easy to Clean

The Exos brace is designed for improved hygiene, allowing:

- Patients to hand wash and dry as needed
- Antimicrobial treated material to reduce odor
- Brace to be removed in order to clean skin or injury site



X-ray without brace



X-ray with brace



It's See Through

The Exos brace is **radiolucent**, allowing:

- X-rays to be taken without removal of brace
- Clinical assessment of fracture healing without the need for re-casting



*Limitations and precautions apply. See product instructions for use.

Reformable, Removable Bracing System



Short Thumb Spica II



Hand Based Ulnar Gutter Brace



Humeral Fracture Brace



Hand Based Radial Gutter Brace



Extended Short Thumb Spica



Pediatric Short Arm Fracture Brace - Open Thumb

What's Your Style?



Camo



Polka Dot



Black



Pink



Spider

Wrist Brace with BOA



Radial Gutter Fracture Brace



Short Arm Fracture Brace



Long Thumb Spica II



Short Arm Fracture Brace - Open Thumb



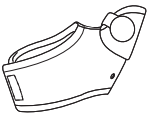
Boxer's Fracture Brace



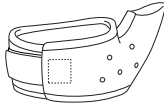
Thumb Spica Fracture Brace




Short Thumb Spica II

	Left	Right	Size
Black Only	NA	NA	XXSmall
	212-31	212-32	XSmall
	212-41	212-42	Small
	212-51	212-52	Medium
	212-61	212-62	Large
	212-71	212-72	XLarge

Short Thumb Spica

	Left	Right	Size
Black Only	210-21	210-22	XXSmall
	210-31	210-32	XSmall
	210-41	210-42	Small
	210-51	210-52	Medium
	210-61	210-62	Large
	210-71	210-72	XLarge

Extended Short Thumb Spica

	Left	Right	Size
Black Only	NA	NA	XXSmall
	211-31	211-32	XSmall
	211-41	211-42	Small
	211-51	211-52	Medium
	211-61	211-62	Large
	211-71	211-72	XLarge

Colors

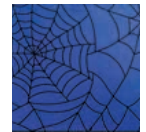
Part #
1111 = Black
4444 = Pink
2293 = Spider
2388 = Camo
3285 = Polka Dot



1111 = Black



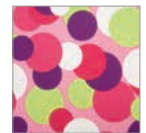
4444 = Pink



2293 = Spider

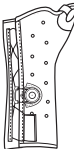


2388 = Camo

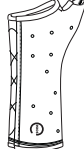


3285 = Polka Dot

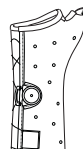
Long Thumb Spica II with Boa

	Left	Right	Size
Black Only	NA	NA	XXSmall
	232-31	232-32	XSmall
	232-41	232-42	Small
	232-51	232-52	Medium
	232-61	232-62	Large
	232-71	232-72	XLarge

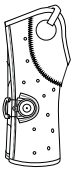
Long Thumb Spica

	Left	Right	Size
Black Only	NA	NA	XXSmall
	230-31	230-32	XSmall
	230-41	230-42	Small
	230-51	230-52	Medium
	230-61	230-62	Large
	230-71	230-72	XLarge


Long Thumb Spica with Boa

	Left	Right	Size
Black Only	231-21	231-22	XXSmall
	231-31	231-32	XSmall
	231-41	231-42	Small
	231-51	231-52	Medium
	231-61	231-62	Large
	231-71	231-72	XLarge


Wrist Brace with Boa

	Left	Right	Size
Color options for sizes XXSmall - Small	221-21	221-22	XXSmall
	221-31	221-32	XSmall
	221-41	221-42	Small
	221-51	221-52	Medium
	221-61	221-62	Large
	221-71	221-72	XLarge

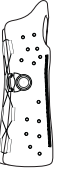
Wrist Brace

	Left	Right	Size
Black Only	NA	NA	XXSmall
	220-31	220-32	XSmall
	220-41	220-42	Small
	220-51	220-52	Medium
	220-61	220-62	Large
	220-71	220-72	XLarge

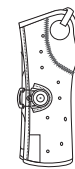
Boxer's Fracture Brace

	Left	Right	Size
Black Only	NA	NA	XXSmall
	325-31	325-32	XSmall
	325-41	325-42	Small
	325-51	325-52	Medium
	325-61	325-62	Large
	325-71	325-72	XLarge


Short Arm Fracture Brace

	Left	Right	Size
Color Options for sizes XSmall - Small	NA	NA	XXSmall
	310-31	310-32	XSmall
	310-41	310-42	Small
	310-51	310-52	Medium
	310-61	310-62	Large
	310-71	310-72	XLarge

Short Arm Fracture Brace-Open Thumb


	Left	Right	Size
Color Options for sizes XSmall - Small	NA	NA	XXSmall
	312-31	312-32	XSmall
	312-41	312-42	Small
	312-51	312-52	Medium
	312-61	312-62	Large
	312-71	312-72	XLarge

Radial Gutter Fracture Brace

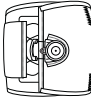
	Left	Right	Size
Black Only	NA	NA	XXSmall
	326-31	326-32	XSmall
	326-41	326-42	Small
	326-51	326-52	Medium
	326-61	326-62	Large
	326-71	326-72	XLarge

Black available in all sizes. Some color options available in pediatric sizes. See product chart for details.

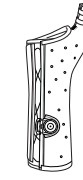
Pediatric Short Arm Fracture Brace Open Thumb

	Left	Right	Size
Color Options	311-11	311-12	3XSmall
	311-21	311-22	2XSmall
	311-31	311-32	1XSmall


Humeral Fracture Brace

		Size
Black Only	330-40	Small-Universal
	330-50	Medium-Universal
	330-60	Large-Universal


Thumb Spica Fracture Brace

	Left	Right	Size
Black Only	NA	NA	XXSmall
	315-31	315-32	XSmall
	315-41	315-42	Small
	315-51	315-52	Medium
	315-61	315-62	Large
	315-71	315-72	XLarge

Hand Based Ulnar Gutter Brace

	Left	Right	Size
Black Only	NA	NA	XXSmall
	125-31	125-32	XSmall
	125-41	125-42	Small
	125-51	125-52	Medium
	125-61	125-62	Large
	125-71	125-72	XLarge

Hand Based Radial Gutter Brace

	Left	Right	Size
Black Only	NA	NA	XXSmall
	126-31	126-32	XSmall
	126-41	126-42	Small
	126-51	126-52	Medium
	126-61	126-62	Large
	126-71	126-72	XLarge



DJO, LLC | A DJO Global Company

T 800.336.6569 D 760.727.1280 F 800.936.6569

1430 Decision Street | Vista, CA 92081-8553 | U.S.A.

DJOglobal.com

Product not made with natural rubber latex.