
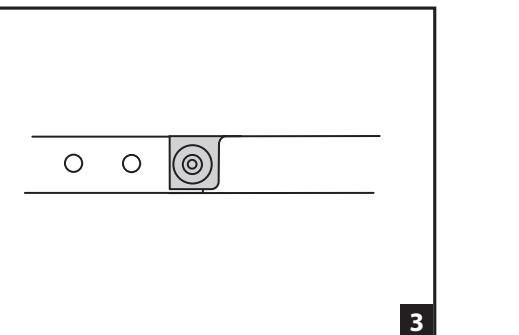
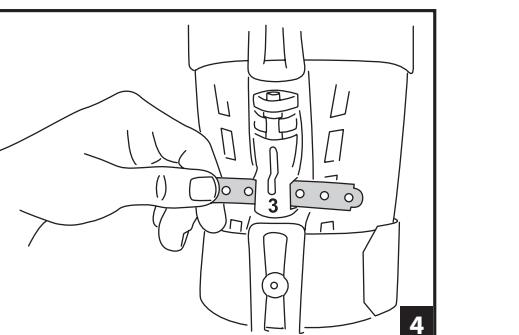
1, 2


3

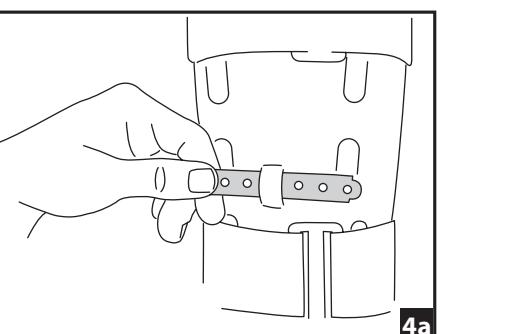
4

4a

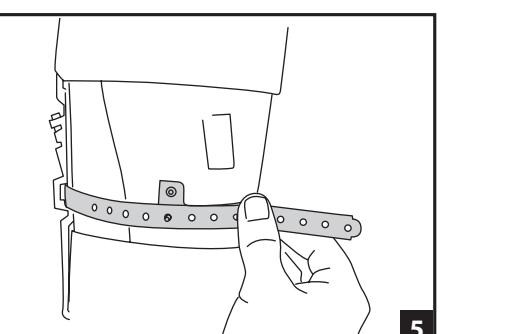5

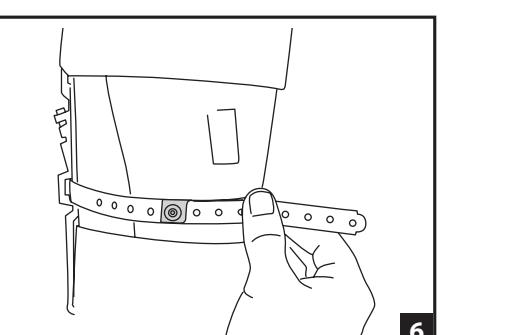6

**ENGLISH**

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

**INTENDED USER PROFILE:**

The intended user should be a licensed medical professional; the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

**INTENDED USE/INDICATIONS:**

The Aircast Brace-Lok is designed for use with the XP Diabetic Walker and Pneumatic Walker to monitor patient compliance. Providing protection.

**CONTRAINdications: N/D****CONTRAINDICATIONS: N/A****ADVERTENCIES Y PRECAUCIONES:**

- Do not use over open wounds.
- Do not modify the device or use the device other than intended.
- Do not use this device if it is damaged and/or the packaging has been opened.
- Do not use this device on patient's incapable of communicating physical discomfort.
- If you develop an allergic reaction and/or experience itchy, red skin after coming into contact with any part of this device, please stop using it and contact your healthcare professional immediately.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, consult your medical professional immediately.

**NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

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- Separate two strips from Brace-Lok sheet.
- Tear off straight end of each strip, leaving a rounded edge (Fig. 1). Each strip has a circular perforation for ease of tearing.
- Connect strips together. Insert white rivet at end of one strip into the first hole at the rounded edge of the second strip (Fig. 2).
- Fold tab over strip, aligning tab lock with tip of white rivet (Fig. 3). Firmly push tab lock onto white rivet until it "clicks". The two strips are now one Brace-Lok.

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**Aplicar Brace-Lok to Walker:****XP Diabetic Walker**

- Insert round end of Brace-Lok through raised opening on back panel of Walker. Opening is located by the center aircell valve, behind the number "3" (Fig. 4).

**Pneumatic Walker for Diabetics**

- Insert round end of Brace-Lok through raised opening on front panel of brace (Fig. 4a).
- Snugly wrap Brace-Lok around Walker. Insert white rivet in hole of Brace-Lok (Fig. 5).
- Fold over tab, aligning tab lock with tip of white rivet (Fig. 6). Firmly push tab lock onto white rivet until it "clicks". The Brace-Lok is now secured to Walker.

Excess Brace-Lok length can be trimmed with scissors. To remove Brace-Lok, cut with scissors.

**CLEANING INSTRUCTIONS:** The Brace Lok may be hand-washed in cold water with mild detergent and air-dried.

**WARRANTY:** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

**NOT MADE WITH NATURAL RUBBER LATEX.**

**NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.**

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