







# Knee Pro Smart Wrap™ Quick Start Guide

## Setting Up the Kelvi Pro<sup>e</sup> Console

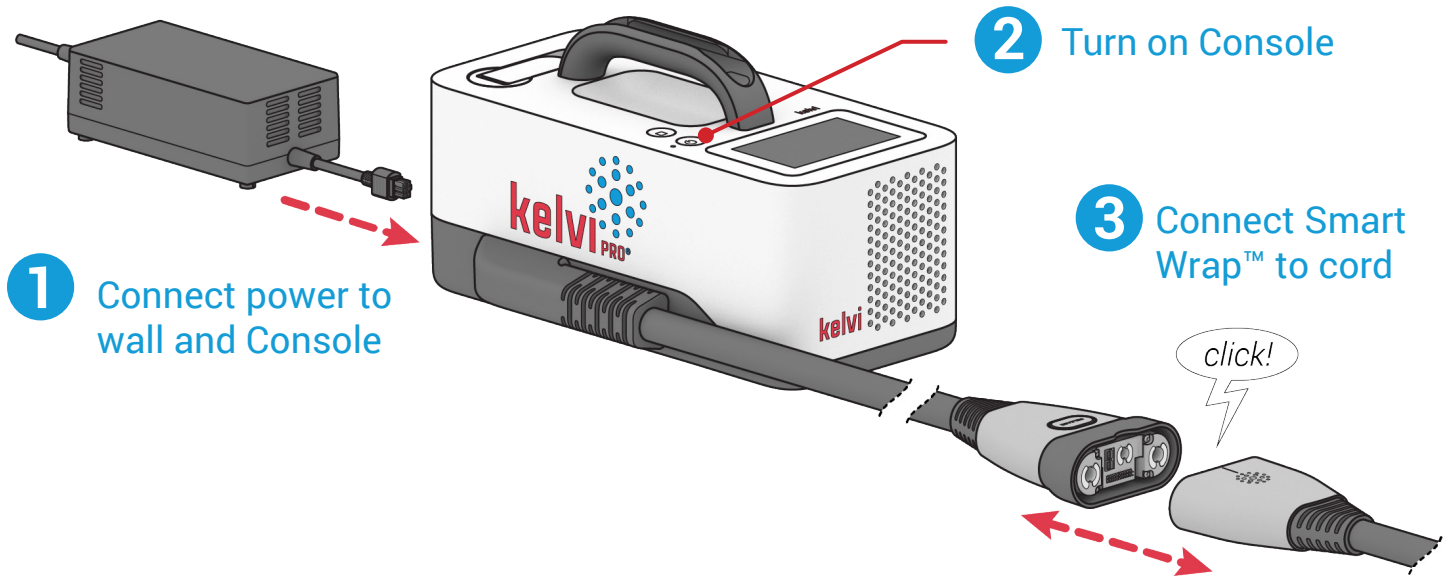

## Applying the Smart Wrap™

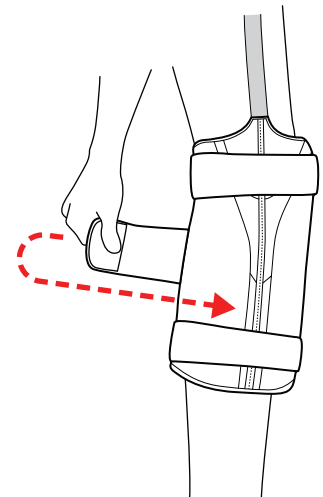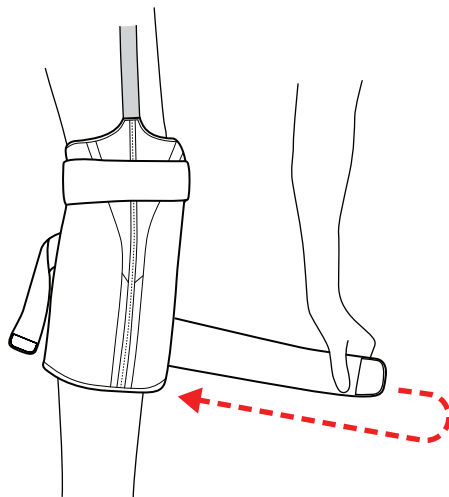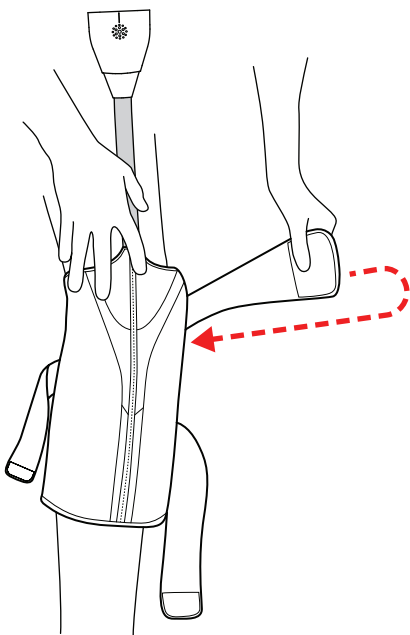


**4** Place the Smart Wrap™ on the leg, centered above the kneecap. Pull and secure top strap.

**5** Secure bottom strap.

**6** Secure middle strap.  
Adjust straps as needed for comfort.


# Setting Up Treatment\*

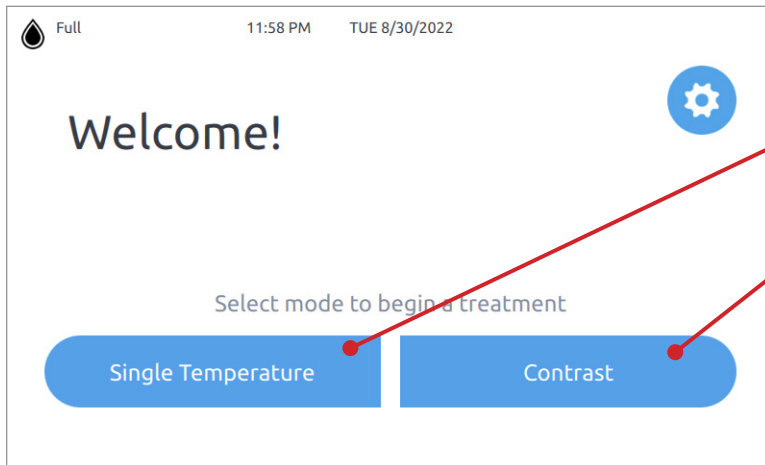

## 7 Select treatment type

On the touchscreen, select **Single Temperature** for constant treatment

Select **Contrast** for alternating cool and hot treatment

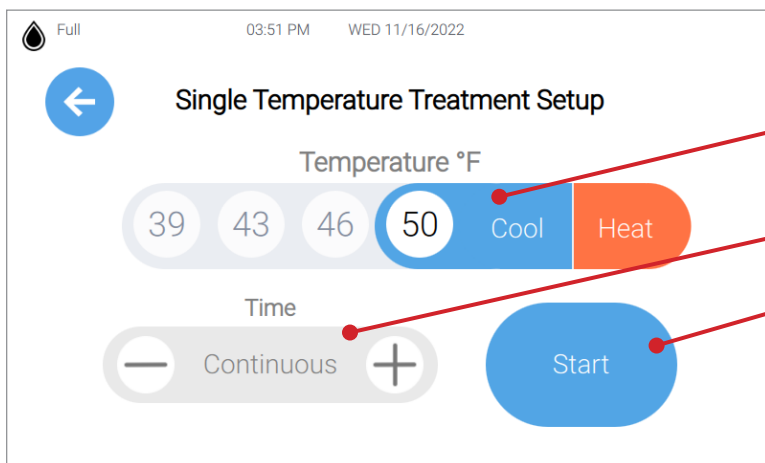


## 8 Set temperature and time

Select **Cool** or **Heat**, then select temperature.

Set a time limit or choose **Continuous**.

Press **Start** to begin treatment.

Smart Wrap™ may be adjusted as needed for comfort during treatment.

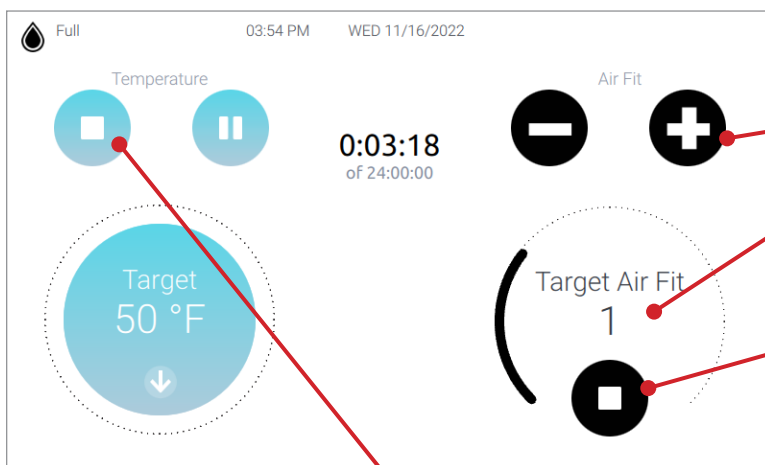

## 9 Adjust air pressure for fit

Select **+** or **-** to change air pressure.

Start low and increase by one level at a time. Do not exceed comfortable pressure.

Press **Stop** to release air pressure.

Press **Stop** to end treatment.

\* Follow clinician prescription for treatment



Scan here for complete instructions for use



**IMPORTANT:** Read complete instructions including indications, contraindications, cautions, and warnings before using this product. For assistance in using the Kelvi Pro™ System, contact your licensed or certified healthcare professional.

