PROCARE®

TRANSITIONAL 172 CERVICAL COLLAR

Patient Instructions

APPLICATION & CARE INSTRUCTIONS:

BEFORE USING THIS MEDICAL DEVICE. PLEASE READ THE INSTRUCTIONS FOR USE COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE. THIS PRODUCT IS INTENDED TO BE USED UNDER THE SUPERVISION OF A HEALTHCARE PROFESSIONAL.

PATIENT APPLICATION:

- 1 Attach the front panel and the back panel with the hook and loop on one side. While sitting or standing and with the head in neutral position (ie: looking forward), slide the front portion of the collar up the neck and properly position the chin support. (fig. 1a)
- Hold the front panel firmly with one hand. Wrap the back-panel 2 around the back of the neck and attach the hook and loop strap to the front panel. (fig. 1b)
- To remove the brace undo the hook and loop on one side only for 3 re-application ease.



(CLEANING & STORAGE INSTRUCTIONS) Hand wash in warm water (30° C, 86°F) using mild soap, rinse thoroughly. AIR DRY.

Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate materials.

WARRANTY:

DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. Current legal regulations apply

FOR SINGLE PATIENT USE ONLY.

Rx ONLY

NOT MADE WITH NATURAL RUBBER LATEX

WARNINGS AND PRECAUTIONS:

If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

Individual results may vary. Neither DJO, LLC nor any of its subsidiaries dispense medical advice. The contents of this

poster do not constitute medical, legal, or any other type of professional advice. Rather, please consult your healthcare professional for information on the courses of treatment, if any, which may be appropriate for you.









DJO, LLC | A DJO Global Company T 800.336.6569 **D** 760.727.1280 **F** 800.936.6569