AIRCAST®

AIRCAST®
AIRSELECT™ ELITE

ENGLISH 2

ESPAÑOL..... 4

DEUTSCH 6

ITALIANO 8

FRANÇAIS..... 10

NEDERLANDS 12

PORTUGUÊS..... 14

ČEŠTINA 16

SLOVENČINA 18

DANSK 20

SVENSKA..... 22

SUOMI 24

NORSK..... 26

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





DJO, LLC
5919 Sea Otter Place
Suite 200
Carlsbad, CA 92010
U.S.A.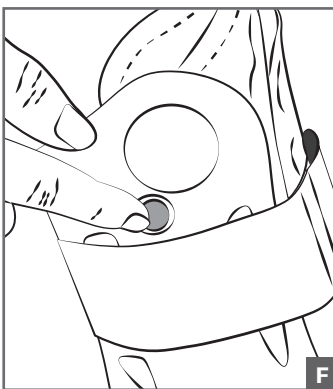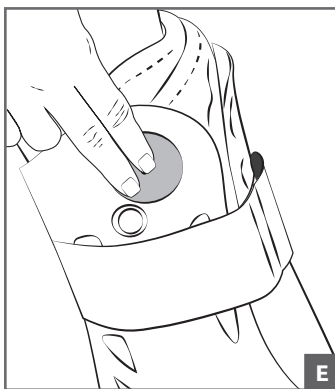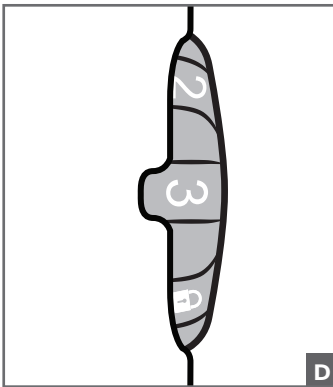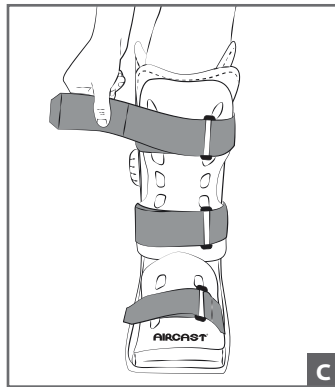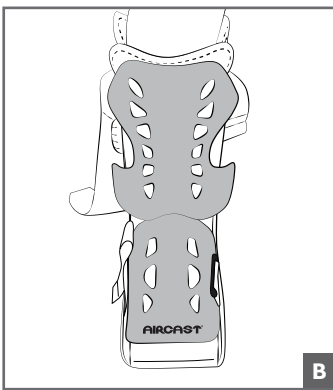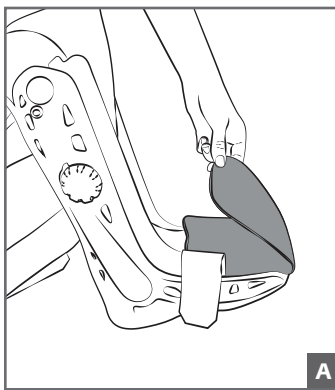

ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

INTENDED USE / INDICATIONS:

The Aircast AirSelect Elite walker is designed to provide support and immobilization to the lower leg, ankle and foot. It may be suitable for use following stable ankle and/or foot fractures and severe lateral ankle sprains. Providing immobilization or controlled movement of the limb or body segment.

CONTRAINDICATIONS:

This device is contraindicated for unstable fractures of the lower leg, ankle and foot. Do not use if you are allergic to any of the materials contained within this product.

WARNINGS AND PRECAUTIONS:

- This device is to be used under the supervision of a healthcare professional. The determination of when to apply the brace and the frequency and duration of use should be strictly at the discretion of the treating physician. Always consult with your physician before making changes to the brace.
- Do not use this device on patient's incapable of communicating physical discomfort.
- Do not over-inflate aircells. Improper aircell inflation may cause significant skin irritation, especially in people with diabetic neuropathy. Reduce aircell pressure with any visual skin changes or reported discomfort.
- Patient's without sensation (i.e. post-op anesthesia, neuropathies, etc.) should be monitored frequently for skin damage, skin irritation or wound management. Use caution when walking on slippery or wet surfaces to avoid injury.
- Do not use over open wounds.
- Do not over tighten straps. This may result in reduction of blood flow or sensation.
- Do not modify the device or use the device other than intended.
- At high altitudes the aircells will expand beyond their optimal level. Adjust aircell support by tightening or loosening straps.
- Do not use this device if it is damaged and/or the packaging has been opened.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, consult your medical professional immediately.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INFORMATION:

1) PREPARE BRACE –

- Unfasten straps, remove front panel, and open toe cover and foot flaps.

2) APPLY BRACE –

- First, put on sock included with the brace.
- While seated, place foot in brace positioning the heel against the back of the brace. Wrap foot flaps over the foot and the toe cover around toes (Fig. A).
- *Replace front panel:* Position front panel so the Aircast logo is over the toes and the foot panel is situated inside the brace. The upper part of the panel should be secured outside the brace (Fig. B).
- Secure straps from bottom to top (Fig. C). Tighten until snug and comfortable.

3) ADJUST AIRCELL SUPPORT –

The ELITE walking brace aircells are initially deflated

- Begin inflation on injured side first. Turn the SELECTOR found on the right-side of the brace to the appropriate number (Fig. D). Number 1 inflates the aircell on the right-side, Number 2 inflates the aircell on the left-side and Number 3 inflates the back aircell. The "lock" position should be used once all aircells are inflated. This prevents accidental deflation.
- To inflate aircell, place finger on top of **large** inflation pump found on the top right side of brace (Fig E). Pump until aircell feels snug.
- To inflate other aircells, rotate SELECTOR to the appropriate number and then press and release large inflation pump. Pump until aircell feels snug. When all aircells are inflated, rotate SELECTOR to "lock" position.
- In the event of over-inflation, deflate aircells.

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4) DEFLATE AIRCELLS –

- To deflate aircells, turn the SELECTOR found on the right-side of the brace to the proper aircell number (Fig. D).
- Press and hold finger on the **small** release button found on the top right-side of brace (Fig. F).
- Deflate until pressure is comfortable.

NOTE: Deflate aircells **before removing brace** for proper fit when reapplying.

NOTE: Higher pressure provides more support when walking, lower pressure is more comfortable when sitting or reclining. Pressure can also be adjusted by tightening or loosening straps.

NOTE: The shell can be trimmed with a cast saw or cast scissors to accommodate larger calves.

CLEANING INSTRUCTIONS:

THE FOAM LINER MAY BE HAND-WASHED IN 86°F/30°C WATER WITH MILD DETERGENT AND AIR-DRIED. DO NOT PUT IN DRYER OR USE OTHER HEAT SOURCE TO DRY.

WARRANTY:

DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOT MADE WITH NATURAL RUBBER LATEX.

INTENDED FOR USE ON A SINGLE PATIENT.

RX only.

MATERIAL COMPONENTS

Shell: POLYPROPYLENE

Front Panel: HIGH DENSITY POLYETHYLENE

Straps: NYLON

Lined Aircells (Anterior and Posterior): POLYURETHANE w/ POLYESTER FABRIC

Inflation Pumps: TPU (THERMOPLASTIC POLYURETHANE)

Liner: NYLON/POLYURETHANE/POLYESTER (OUTER/FOAM/INNER MESH)

Binding: POLYESTER

Outer Sole / Insole: EVA/RUBBER

Selector dial: NYLON

Sock (Elite): ACRYLIC/COTTON

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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