

MalleoForce®

PATIENT APPLICATION



BEFORE USING THIS DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE:

THE USER SHOULD BE ABLE TO:

- READ AND UNDERSTAND THE DIRECTIONS, WARNINGS AND CAUTIONS.

INTENDED USE/INDICATIONS:

Ankle support with padding for support and compression

INDICATIONS:

- Post-operative or post-traumatic use, chronic soft tissue inflammation in the ankle area
- Feelings of instability
- Sprains
- Ligament injuries, ligament weaknesses
- Load-related pain in the foot and ankle
- Joint effusions
- Swelling
- Mild Osteoarthritis

CONTRAINDICATIONS

There are no known contraindications.

The MALLEOFORCE® Ankle Support should only be used after consulting a health care professional if you suffer from the following symptoms:

- Skin diseases and injuries in the ankle area (e.g. allergy-related skin changes, inflammation, scars with swelling, redness, hyperthermia)
- Varicose veins with reflux disorders
- Lymphatic disorders, non-specific soft tissue swelling
- Circulatory or sensory disorders in the legs

WARNINGS AND PRECAUTIONS:

The product is intended for single patient use only. If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.

CARE:

- Hand wash in lukewarm water (max. 30 °C) with a mild detergent
- Air dry (do not place on radiators/heaters or other heat sources)
- Do not use bleach or chlorine, do not iron.



Application Information

MALLEOFORCE®

1. Sit down to fit the MALLEOFORCE/MALLEOFORCE PLUS.
2. Roll down the ankle support from the top blue border as far as the silicone pads.
3. Then grip the support by the pads and pull it over your foot.
4. The support is properly fitted when the heel sits firmly in place and the lateral pads are positioned behind and below the ankle.
5. Pull up the upper end of the support again and straighten it over the foot and ankle.

INTENDED FOR USE ON A SINGLE PATIENT

WARRANTY:

DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

COMPOSITION:

Knitted material: polyamide, elastin, polyester / Pad: silicone / Cover: terrycloth / Strap: polyamide, polyurethane, polyester, natural rubber, elastane, polypropylene



CAUTION: THIS PRODUCT CONTAINS NATURAL RUBBER LATEX WHICH MAY CAUSE ALLERGIC REACTIONS.



DJO, LLC

T 800.336.6569 D 760.727.1280 F 800.936.6569
2900 Lake Vista Drive | Lewisville, TX 75067 | U.S.A.
DJOglobal.com