

VersaROM™ Hip

PATIENT APPLICATION



BEFORE USING THIS DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE:

The user should be able to:

- Read and understand the directions, warnings and cautions.

INTENDED USE/INDICATIONS:

Post-Op use following minimally invasive hip procedures; Labral repair; with or without gluteus medius repair; ROM control following primary or hip revision surgery; Post-op use following proximal hamstring repair.

CONTRAINDICATIONS:

Pulmonary, cardiovascular or skeletal conditions which have risk to be made worse as a result of compression and/or pressure, unstable, displaced fractures.

WARNINGS AND PRECAUTIONS:

If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

CLEANING INSTRUCTIONS:



Hand wash in water (86oF/30oC) using mild soap. Air dry.

NOTE: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.



Do NOT wash.



Do NOT iron.



Do NOT tumble dry.



Do NOT bleach.

MATERIAL CONTENTS:

Mesh: Nylon; **Spacer:** Polyester; **Binding:** Poly/Cotton

WARRANTY:

DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.





RX ONLY.

INTENDED FOR SINGLE PATIENT USE.

NOT MADE WITH NATURAL RUBBER LATEX.

Application Information

FOR SIZING AND ADJUSTMENT INSTRUCTIONS, please reference the enclosed IFU.

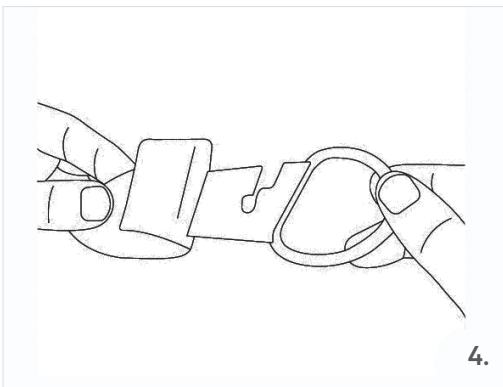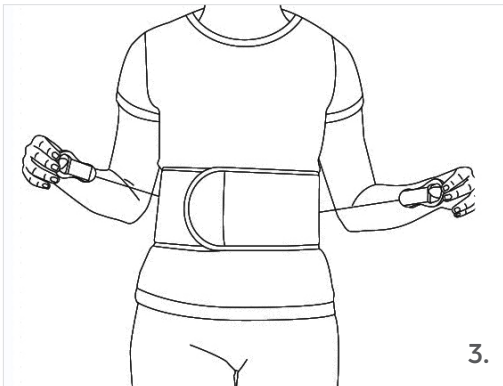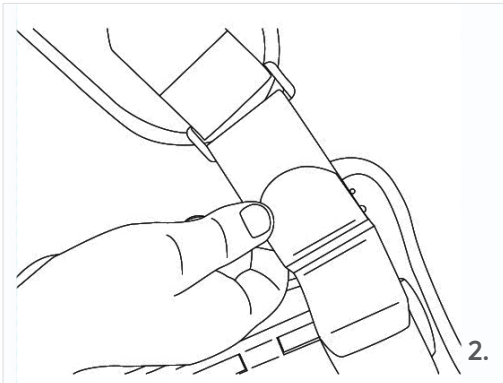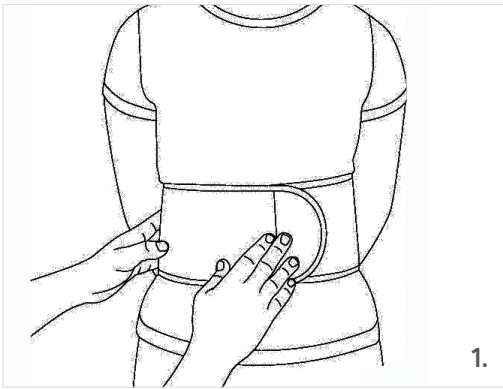

1. Wrap the waist belt approximately 1" (2.5cm) above the widest part of the hips and secure the hook and loop attachment on the front side of the patients.
2. Wrap thigh strap around leg and adhere hook and loop tab.
NOTE: To adjust the thigh strap, peel open the shiny tab to expose the strap ends. Cut the strap and reapply the hook tab. Please be careful not to cut the strap too short.
3. Pull the pull tabs around and attach to the front of the waist belt.

ADJUSTING PULL TAB LENGTH (OPTIONAL)

4. Remove lace spool from the pull tab pocket and wind lace around the spool to shorten the length of the pull tab. Unwind lace from the spool to lengthen the pull tab.

BRACE REMOVAL

5. To remove the hip brace, detach the pull tabs on the side of the waist belt then, unfasten the belt and thigh cuff. Remove hip brace.

NOTICE:

WHILE EVERY EFFORT HAS BEEN MADE IN STATE OF THE ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THIS DEVICE IS ONLY ONE ELEMENT IN THE OVERALL TREATMENT PROGRAM ADMINISTERED BY A MEDICAL PROFESSIONAL. THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.



DJO, LLC

T 800.336.6569 D 760.727.1280 F 800.936.6569
2900 Lake Vista Drive | Lewisville, TX 75067 | U.S.A.
DJOglobal.com

COPYRIGHT © DJO, LLC 2020

MKTIS00-9964 Rev A

Individual results may vary. Neither DJO Global, Inc. nor any of its subsidiaries dispense medical advice. The contents of this document do not constitute medical, legal, or any other type of professional advice. Rather, please consult your healthcare professional for information on the courses of treatment, if any, which may be appropriate for you.