ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USE/INDICATIONS: Mild to Moderate uni-compartmental (one compartment) Knee Osteoarthritis (degenerative arthritis) or other cartilage injury/surgery. Post-operative Rehabilitation.

INTENDED USER PROFILE: This device is intended for users who are physically and mentally able to perform the instructions described in this IFU.

CONTRAINDICATIONS: None

WARNINGS AND PRECAUTIONS: If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

APPLICATION INFORMATION:

1. Unfasten straps and flaps so the brace is completely open.
2. Align the center of hinge with the center of the knee.

- NOTE:** The hinge bar should be located on the unaffected side of the knee.
3. Wrap the thigh flap first then the calf flap to secure brace. Adjust for a snug fit.
 4. Thread the straps through the D-Rings and tighten.

Hinge Adjustment:

5. Rotate both knobs on hinge clockwise to increase off-load. Ensure the numbers are increasing as you turn the knobs. Adjust as needed to help relieve OA symptoms.

NOTE: Ensure no more than a 2-digit difference.

CLEANING INSTRUCTIONS:

Hand wash with soap in 30°C water.

Do not bleach. Do not iron.

MATERIAL CONTENTS: Polyester, Aluminum, Nylon, PVC, Carbon, Polycarbonate, Suede, Silicone

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

Rx ONLY.
FOR SINGLE PATIENT USE ONLY.
NOT MADE WITH NATURAL RUBBER LATEX.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

CAUTION: FEDERAL LAW (U.S.A.) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PROFESSIONAL.

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