




Spine Bracing System

PATIENT APPLICATION INSTRUCTIONS

EXOS® FORM™ II 631 / 637

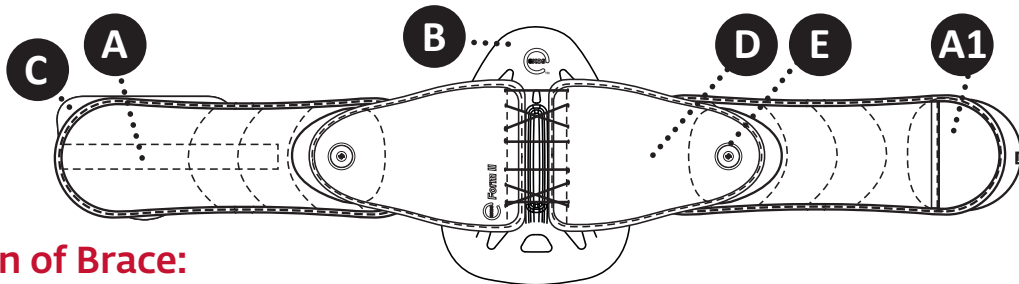

Application of Brace:

1 **LOOSEN BRACE** by pulling out on both Boa® Knobs (E) and pull Rear Panel Section (D) in opposite directions.



2a **APPLY BRACE** by inserting hand into the Donning Mitt pocket (A1) and wrapping the Belt Wings (A) around the waist.



2b **NOTE:** Center Tall Rear Panel (B) on back. The Exos logo on the Front Panel (C) should be facing upright and away. The point of the Tall Rear Panel should be facing up.



3 **TIGHTEN BRACE** by pushing in both Boa® Knobs and turn knobs clockwise.



NOTE: Left Boa® Knob controls top half of brace. Right Boa® Knob controls bottom half.

OPTIMAL COMPRESSION: When sitting down, prior to sitting, release compression by pulling-out on Boa® Knobs. Once seated, tighten compression by pushing in on Boa® Knobs. When standing up, tighten compression if necessary.

BRACE REMOVAL: Loosen brace compression by pulling-out on Boa® Knobs. Detach hook and loop connection on Belt Wings and remove brace.

USE AND CARE (CLEANING & STORAGE INSTRUCTIONS): Hand wash in cold water, less than 86°F (30°C), using mild soap. Air dry. **NOTE:** If not rinsed thoroughly, residual soap may cause irritation and deteriorate material. Do NOT iron. Do NOT expose to heat over 120°F (49°C). Do NOT tumble dry. Do NOT bleach.

MATERIAL CONTENTS: Nylon, Polyethylene, Polyester, Antimicrobial Treated Material, Polyurethane, Stainless Steel, Polycarbonate, Thermoplastic Elastomer, Polyoxymethylene, Carbon Steel, Acrylic Adhesive. **FOR SINGLE PATIENT USE ONLY. NOT MADE WITH NATURAL RUBBER LATEX.**

CONTRAINDICATIONS: Pregnancy and circulation, pulmonary, cardiovascular or skeletal conditions which have risk to be made worse as a result of compression and/or pressure.



For more information including application videos, please visit DJOGlobal.com/products/exos/exos-form-ii-back-brace or call **888.405.3251**

BEFORE USING THIS MEDICAL DEVICE, PLEASE READ THE INSTRUCTIONS FOR USE COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

*Boa® is a registered trademark of Boa Technology, Inc.

Patented

Copyright © 2019 by DJO, LLC

Individual results may vary. Neither DJO Global, Inc. nor any of its subsidiaries dispense medical advice. The contents of this brochure do not constitute medical, legal, or any other type of professional advice. Rather, please consult your healthcare professional for information on the courses of treatment, if any, which may be appropriate for you.

MKTISoo-8776 Rev A

Next Appointment:

Date: _____

Time: _____

